

PSL classic

Quick guide to controls

50 72

Rotate eyepiece bodies to adjust PD

LED fixation targets

Click and hold the trigger to turn the lamp illumination on

2 x

Double click and hold the trigger to turn lamp illumination and LED fixation targets on

Slit Wheel

1x1 0.15 0.5 0.8 1.6 12

Filter Wheel - Red free, Blue, Neutral density (0.8) and Clear

Rotate eyepieces to focus the oculars

16x 10x

Magnification change lever

Rotate wheel to adjust brightness

This LED comes on when charging

The battery lasts for more than 50 minutes

This LED flashes when the battery is almost empty

The battery can be charged by the docking station or device